

EZ CLAMP® UMBILICAL CORD CLAMP

REVOLUTIONIZES THE WAY UMBILICAL CLAMP OPERATES

A unique Sterile Umbilical Cord Clamp that combines two clamps and simultaneously cuts in between, in one simple and safe action.

DESCRIPTION

The EZ Clamp is composed of two regular Umbilical Cord Clamps and a Stainless Steel Knife that are combined together into one instrument which looks, feels and operates as a regular Umbilical Cord Clamp. The EZ Clamp will separate after use into two parts - one will be a regular Umbilical Cord Clamp, and the second part with the knife will clamp the placenta.

ADVANTAGES

- Increases safety for infant and mother.
- convenient grip of the clamp.
- Enables a clear view of the inner veins.
- Convenient blood collecting
- Four traditional stages in one easy action.
- Reduces the danger of cross infection.
- No dead space between clamp and cutting point.
- No need of scalpel or scissor near an infant.
- No risk of blood splashing.
- Simple, faster & cost effective.
- Disposable, for one time use.

HOW IT WORKS

It works the same way as a normal Umbilical Cord Clamp and is ready to use immediately. The user simply presses and closes the clamp (exactly as done with the regular Umbilical Cord Clamp), the clamp will close and cut the cord, then it will automatically separate into two parts - one will be the regular Umbilical Cord Clamp that will stay with the baby and the second part will be attached to the placenta. Each EZ clamp is individually sterile packed in a tray; it is disposable and can be used only once.

INSTRUCTION:

The user should practice the EZ clamp before using it on a baby for the first time.

Check that the white clamps are firmly locked in position.

The word BABY on the EZ clamp should point to the BABY side.

The umbilical cord should be in the middle of the white clamp.

Simply press strongly on the green housing close the clamp and the green housing until both closes and are locked.

We invite you to see a tutorial on www.kencap.us/ez-clamp

CAT. No. 5600 EZ CLAMP STERILE STANDARD MODEL.

CAT. No. 5666 EZ CLAMP STERILE SPECIAL MODEL, Separate into three parts (2 clamps and housing)

10 EZ Clamps per box, 10 boxes per master case (100 EZ Clamps)

Master case size: 65cm. X 25cm. X17cm. (22.5" X 10" X 7")

US patent and others



KENCAP LTD. P. O. Box 124, YAHUD 56101 ISRAEL, kencap@kencapltd.com, www.kencapltd.com
KENYON INDUSTRIES, INC RINGWOOD, NJ. 07456 USA, kenyon@warwick.net, www.kenyonusa.com

www.kencap.us



EZ CLAMP[®]


UMBILICAL CORD CLAMP

Two ways to hold the EZ CLamp:


Option 1:



Option 2:

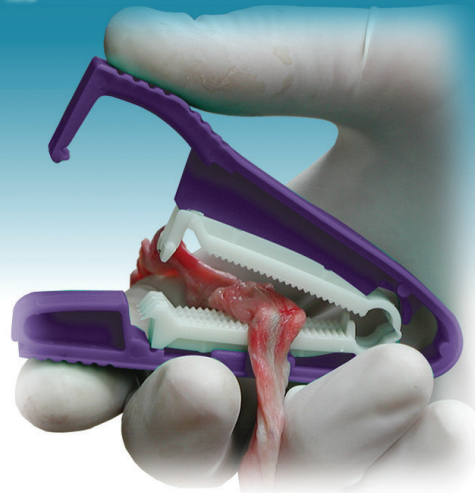


Two models:

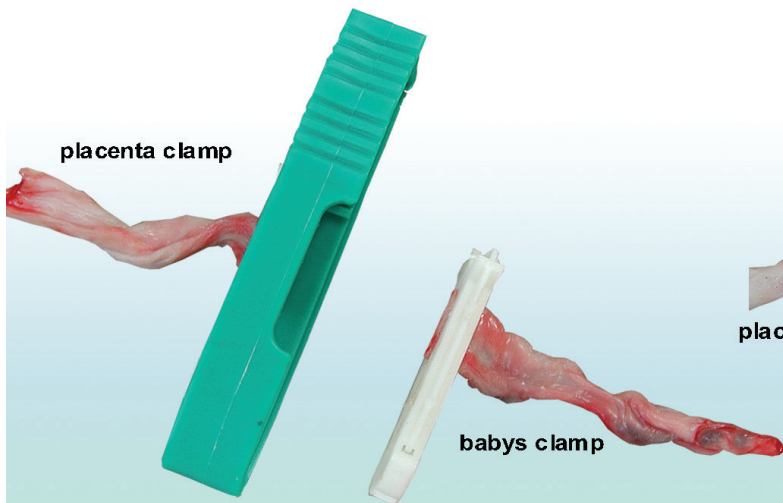

placenta clamp

babys clamp

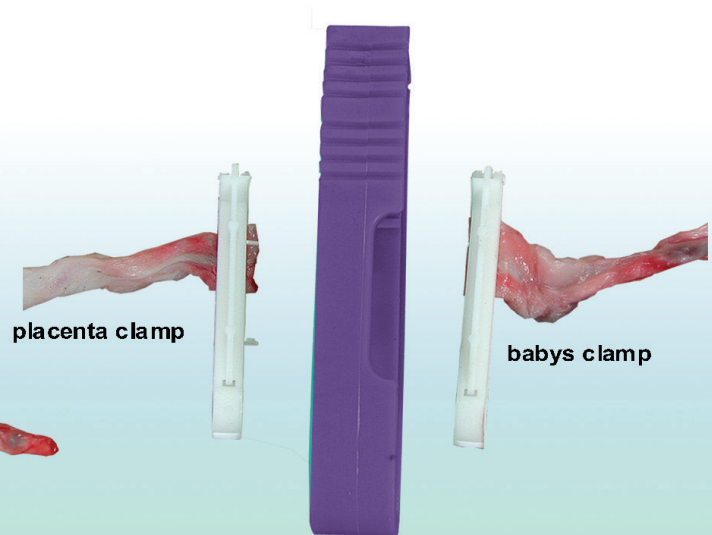

placenta clamp

babys clamp

CAT No. 5600 - Standard.

CAT No. 5666 - 2 Clamps and Housing.

